

# Fecal Float Ready-to-Use



*Fecal Float is a sodium nitrate solution to be used in preparing fecal examinations.*

## Benefits:

- Specific gravity 1.25 to 1.30


## Formulation & Packaging:

- Sodium nitrate in gallon jug

## Directions:

- Put a small amount of fresh feces in small container and add fecal float until full. Place a coverslip on top of the container, making certain that contact is made with the solution. Let stand for 10 minutes, remove coverslip and place on microscope slide. Examine under microscope.

### Labeled For

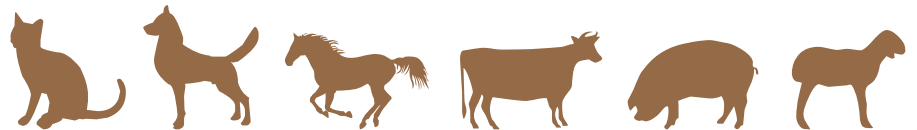

11-2010

**PHOENIX - the RIGHT products at the RIGHT price!**



[www.clipperdist.net](http://www.clipperdist.net)

BROUGHT TO YOU BY  
**CLIPPER**