

Ovum Flotation Dry

Ovum Flotation Dry, when reconstituted is used for fecal flotation analysis where a specific gravity of 1.18 is required.



Benefits:

- Zinc sulfate picks up giardia cysts (without distortion)
- More accurate and provides faster results than other fecal float mediums


Formulation & Packaging:

- 1.32 kg of zinc sulfate powder for reconstitution in each gallon

Directions:

- Place a small amount of fresh feces in a small container. Fill with flotation medium and centrifuge for 5 minutes. Place coverslip on top of container making sure that the solution comes in contact with the coverslip. Remove coverslip, place on microscope slide and examine under microscope.

Labeled For

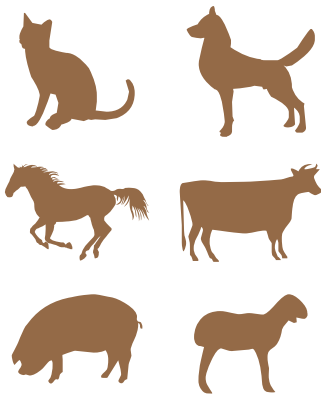

100112

PHOENIX - the RIGHT products at the RIGHT price!

PHOENIX

www.clipperdist.net

BROUGHT TO YOU BY
CLIPPER